

Bruising from those traditional headband N95 masks?

The Solution:
Our STRAPLESS N95 Mask!

No more sore ears!

No more bruised faces!

No more foggy glasses and face shields!




Same Protection As Standard N95 Masks

Like traditional N95 masks, this strapless N95 filters out 95% of 0.3-micron particles in the air while allowing the wearer to breathe freely. The streamlined design and flat profile make this mask less bulky, lighter, and easier to store than other N95 masks.

CHRIS DUDECK

Independent Representative
Carlsbad, California
c.dudeck@pbcsolutions.org
(760) 410-3148

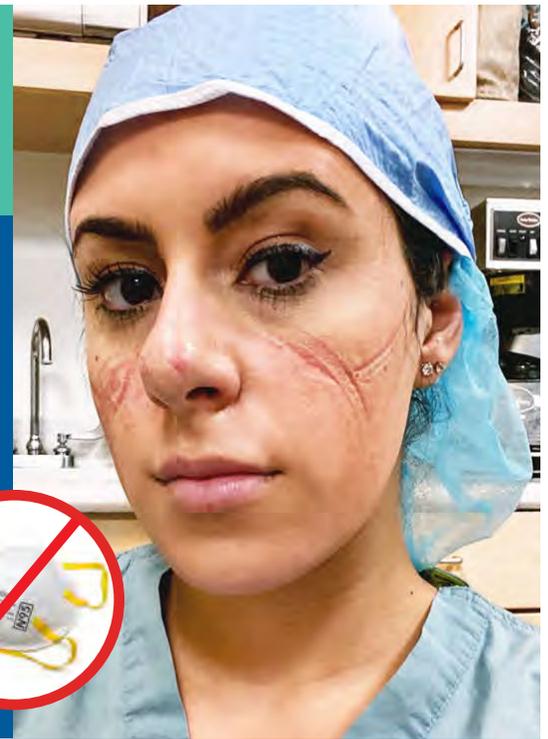

The Comfortable Alternative N95 Mask

This NIOSH-certified strapless N95 mask is unlike any other mask. Unlike traditional N95 masks, this strapless mask creates an airtight seal using medical-grade adhesive, eliminating painful issues caused by tight elastic straps and ties, and metal nose clips.



ALLIANT
BIOTECH

alliantbiotech.com

The Solution: Our STRAPLESS N95 Mask

Save Money on Storage and Shipping

| STRAPLESS N95 MASK | 3M 1860 N95 MASK |
|--|--|
|  |  |
| Each bag has 50 masks Each case has 7 bags/350 masks Each pallet holds 80 cases/28,000 masks 26 pallets fit on a 53' trailer 728,000 masks fit on a 53' trailer | Each box has 20 masks Each case has 6 boxes/120 masks Each pallet holds 50 cases/6,000 masks 26 pallets fit on a 53' trailer 156,000 masks fit on a 53' trailer |

It would take 5 trucks (4.66 to be exact) to ship the same amount of traditional N95 masks compared to our strapless N95 masks. If each mile costs an average of \$4.25 per mile for a 53' trailer, the shipping cost savings (and reduction in carbon emissions) is tremendous.

Application is Easy



PREPARE

Peel off the backing. Press down deep over the bridge of your nose with your fingertips before sliding your fingertips out toward your ears.



POSITION

When sliding fingertips outward under the eyes, swipe in a straight horizontal line across the bottom of your eye socket. Do not use a downward motion.



SECURE

Ensure all areas below the chin and nose bridge are well sealed for a secure fit. Practice applying in a mirror.*



Global Safety First, LLC NIOSH Approval TC#84A-8133
Manufactured by Avery Dennison
*See full IFU for proper application.
MKT-021-Rev1

Application Video

