

# STRAPLESS N95 MASK

## Mask Sizing Tips

To ensure the mask fits comfortably on your face and provides optimal protection, it is important to follow all instructions and to select a properly sized mask.



Below are tips for selecting a properly sized mask:

1. Mask edges should fit outside of your eye-line and not quite to your ears.
2. If you have access to all sizes, it is recommended to try the large size first. If this does not fit properly, try the small or extra large size to find the right fit for you.
3. Ensure an adequate tab can be formed under the chin to allow for a good seal.
4. If the mask is too loose or too tight, the mask will not fit comfortably. The mask should allow you to comfortably speak and move your head during wear.

 us +1 888-307-1144

 email [info@alliantbiotech.com](mailto:info@alliantbiotech.com)

 website [www.alliantbiotech.com](http://www.alliantbiotech.com)



Scan for application instructions.

Designed by Global Safety First, LLC,  
Manufactured by Avery Dennison  
NIOSH Approval TC#84A-8133  
\*See full IFU for proper application.