



[www.signalmd.com](http://www.signalmd.com)

## INSTRUCTIONS FOR USE

Mallet – SMC Product Code: BIO-1-1090

- Mallet tips are to be replaced **ONLY** with Signal Medical Corporation's product (BIO-1-1091). **Use of any other mallet tip will void the warranty.**
- Mallet tips should be replaced when any one of the following occurs:
  - Mushrooming over,
  - Threads are worn down,
  - Other forms of damage or deforming of mallet tip is observed,  
or
  - Mallet tip appears discolored.
- Use only the mallet tip end for driving implants.
- Do not use metal end for striking implants.
- Remove mallet tip before each sterilization, then inspect before reattaching.
- Prior to each use, screw mallet tip down securely, but do not overtighten to avoid stripping of threads or other damage.
- **NEVER** use mallet without mallet tip attached.