

Mg OSTEOCRETE is fully synthetic with an ideal pH level of $7.4 \leq \text{pH} \leq 7.9$.

Mg OSTEOCRETE Basic Kit



Contents:

- Magnesium Powder
- High Viscosity Solution (30 Second Fast-Set Putty)
- Medium Viscosity Solution
- Mixing Spatula
- Basin

Basic Kit Sizes:

- 5cc
- 10cc
- 15cc


Bead Mat Kit




PREPARATION GUIDE

1 Mix

Add powder and High Viscosity Solution in basin. Mix briskly (two revolutions per second) to form a cohesive mixture.

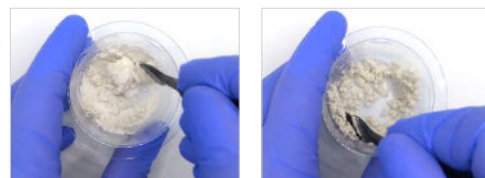

Physically work and press the material until it is integrated into a homogeneous mass.

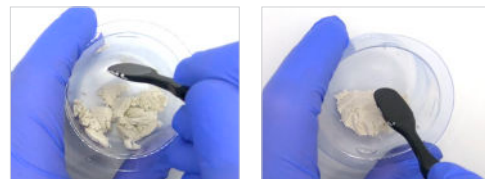

2 Transfer

Move the material from the basin onto the bead mat and use scraper to shape into beads.



Gently bend the bead mat to release the beads to be placed into contained voids.

