

# SOFT




**DryMax Soft is a sterile, conformable, superabsorbent dressing for use on exuding wounds. DryMax Soft absorbs and retains exudate in the dressing. The exudate is absorbed vertically by the dressing and retained in the core forming a gel, thus minimizing the risk of maceration and maintaining a moist wound environment.**


The seamless cover simplifies positioning over the wound. The design allows the dressing to be applied with either side facing the wound. This feature also makes it possible to stack dressings on top of each other if extra absorption capacity is required.

The dressing remains dry and comfortable for the patient. Simultaneously it contributes to a moist wound environment. The hydrophilic contact layer transports fluid effectively away from the wound and distributes it evenly into the ABSORBEST core.


The ultra-thin and highly conformable features allows the dressing to be used on all parts of the body. DryMax Soft is designed to work very well together with compression therapy.

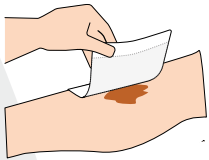

The dressing must be placed close to the wound, or placed in close combination with a suitable dressing.

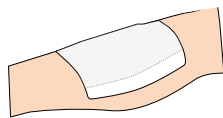

Secure the dressing with suitable bandage, film or tape.

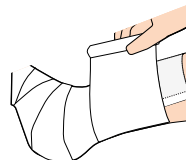

DryMax Soft can be used in conjunction with compression therapy.



| Item                   | REF       | Pcs/box | Product size (cm) | Absorption capacity* |
|------------------------|-----------|---------|-------------------|----------------------|
| DryMax Soft 7.5x7.5 cm | F60011/10 | 10      | 7.5x7.5           | 50 ml                |
| DryMax Soft 10x10 cm   | F60013/10 | 10      | 10x10             | 130 ml               |
| DryMax Soft 10x15 cm   | F60012/10 | 10      | 10x15             | 230 ml               |
| DryMax Soft 10x20 cm   | F60015/10 | 10      | 10x20             | 330 ml               |
| DryMax Soft 15x20 cm   | F60014/10 | 10      | 15x20             | 400 ml               |
| DryMax Soft 20x20 cm   | F60016/10 | 10      | 20x20             | 600 ml               |
| DryMax Soft 20x25 cm   | F60018/10 | 10      | 20x25             | 780 ml               |
| DryMax Soft 20x30 cm   | F60017/10 | 10      | 20x30             | 960 ml               |
| DryMax Soft 37x56 cm   | F60019/10 | 10      | 37x56             | 3000 ml              |

\*) According to international standard EN 13726



Manufactured by Absorbest AB Sweden  
 Distributed by Alliant Biotech, LLC.  
[www.alliantbiotech.com](http://www.alliantbiotech.com)  
[orders@alliantbiotech.com](mailto:orders@alliantbiotech.com)  
 (844) 652-1100



Manufactured by Absorbest AB Sweden  
[www.absorbest.se](http://www.absorbest.se) | [info@absorbest.se](mailto:info@absorbest.se)